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## Instructions of use

The walking stick converts easily into a seat stick. A simple push button mechanism allows the seat to be fixed for sitting on like a bicycle seat. To convert the walking stick to a seat, depress the button on top of the stem, 'flip' into position and the seat will lock into place. As a seat please be aware to keep always a lightly angled position (inclination between 10-15°). To return to a walking stick, depress the button that is now visible in the middle of the seat, 'flip' back and the handle will lock into place. Adjustable Flipsticks Set the height as a walking stick (87-91cm) and as a seat (75-80cm) in the following way 1. Unscrew the collar in the bottom section of the shaft near the tip (ferrule). 2. Push in the button. 3. Find height required by twisting the two shaft sections apart so that the button goes into the next hole, up or down.

4. When correct height is achieved, tighten the collar again. Please ensure that every section is fully tightened before use. Folding Flipsticks. To unfold the flipstick the sections will lock into place when taken out of the bag. A stretchy cord running through the stems makes each section lock together. Check all stems have locked into place before use. To fold back up, pull apart each section and fold into the carry case.

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